

Sinocare

BSX-516



USB Port
Support



WHO Function



Large LCD
Display



Back-Light
Function



Automatic
Turn-off



One-year
Warranty



Oscillometric
Method




Two Users
Memory

Add: No. 265 Guyuan Road Hi-tech Zone, Changsha, Hunan, China, 410205
Tel: 0731-89935929 Email: intl_mkt@sinocare.com Web: www.sinocareintl.com


Technical Specifications

Product Type:	BSX516
Cuff Circumference:	22cm~32cm(± 5 mm)
Memory:	2*99 groups memory (Double users)
Measurement Range:	Blood pressure 0-299mmHg Pulse 40-180 beats/min
Accuracy:	Blood pressure: ± 3 mmHg Pulse: $\pm 5\%$ of the reading
Material:	ABS+LCD digital display
Power Source:	Internal-DC 6V (4*AA) / External-DC6V 0.6A
Operating Conditions:	Temperature: 5°C to 40°C Humidity: 15% to 85% RH Pressure altitude: 80Kpa~106Kpa
Storage And Shipping Conditions:	Temperature: -20°C to 55°C Humidity: $\leq 93\%$ RH Pressure altitude: 70Kpa~106Kpa
Measurement Mode:	Oscillometric method


Use methods and precautions

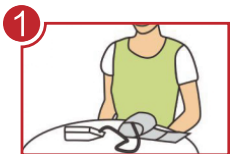


Keep relaxed before measuring
sit down quietly for a moment

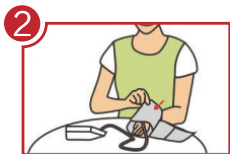

Palms up, put the arm
cuff parallel to the heart

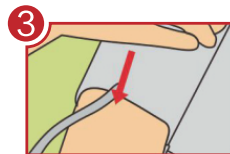


Palms up, keep the intake pipe
and the arteries paralleled

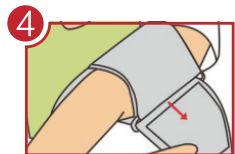


Wrap the arm cuff around
your arm tightly in the opposite
direction, paste together



Place one finger into it
is the most appropriate

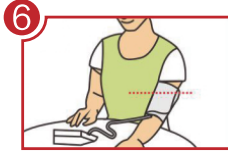

Keep the arm cuff paralleled
to the heart, palms up

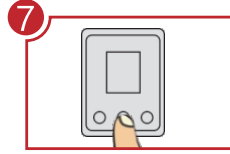

Press the on/off button, keep
relaxed and start measuring



High pressure, low pressure,
pulse instantly displayed after
40 seconds